



CASE REPORT

Nd:YAG

**SYNCHRO REPLA:Y
SYNCHRO FT**



The Code of Excellence

Angioma on the Nose


25 years old female patient
Phototype III

Settings

Fluence	103 J/cm ²
Spot Size	2.5 mm
Nr. of Pulses	1
Pulse Duration	6 ms



Before treatment

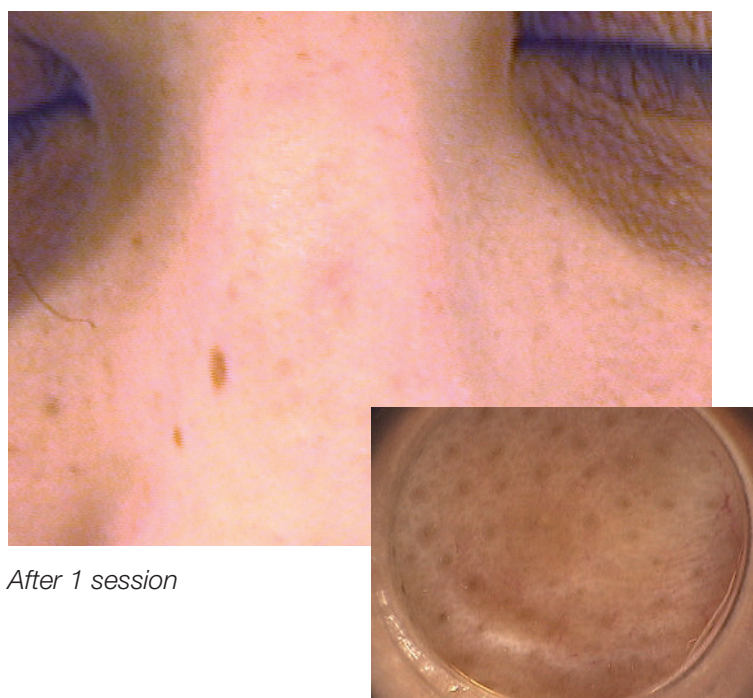

After 1 session

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Nose Wings Teleangectasias

33 years old female patient

Phototype II

Settings

Fluence 100 J/cm²

Spot Size 2.5 mm

Nr. of Pulses 1

Pulse Duration 7 ms



Before treatment



After 1 session

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Nose Teleangectasias

25-years old

Caucasian female patient

Phototype II

Settings

Fluence 115 J/cm²

Spot Size 2.5 mm

Nr. of Pulses 1

Pulse Duration 8 ms



Before treatment
(picture taken using polarized light)



Twenty days after a single treatment
(picture taken using polarized light)

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD

Department of Dermatology

University of Florence, Italy

Nose Teleangectasias

52-years old

Caucasian female patient

Phototype II


Settings

Fluence 115 J/cm²

Spot Size 2.5 mm

Nr. of Pulses 1

Pulse Duration 8 ms

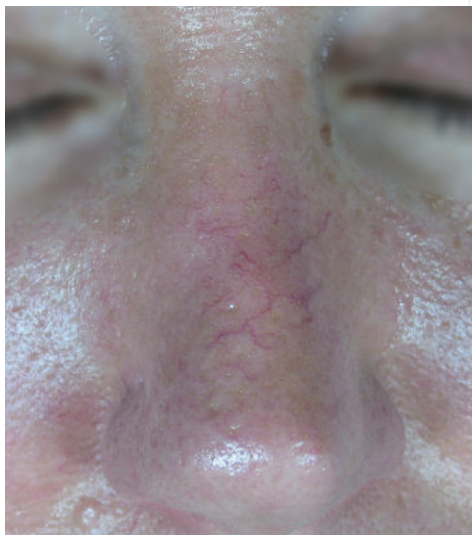

*Before treatment
(picture taken using polarized light)*



Forty-five days after the 2nd treatment

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD

Department of Dermatology

University of Florence, Italy

Face Teleangiectasias

50-years old

Caucasian female patient

Phototype II

Settings

Fluence 115 J/cm²

Spot Size 2.5 mm

Nr. of Pulses 1

Pulse Duration 8 ms



*Before treatment
(picture taken using polarized light)*



*Thirty days after the 2nd treatment
(picture taken using polarized light)*

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD

Department of Dermatology

University of Florence, Italy

Lower Limbs Teleangiectasias

54-years old

Caucasian female patient

Phototype III

Settings

Fluence 120 J/cm²

Spot Size 5 mm


Nr. of Pulses 2

Pulse Duration 5 ms + 18 ms

Delay 20 ms



*Before treatment
(picture taken using polarized light)*

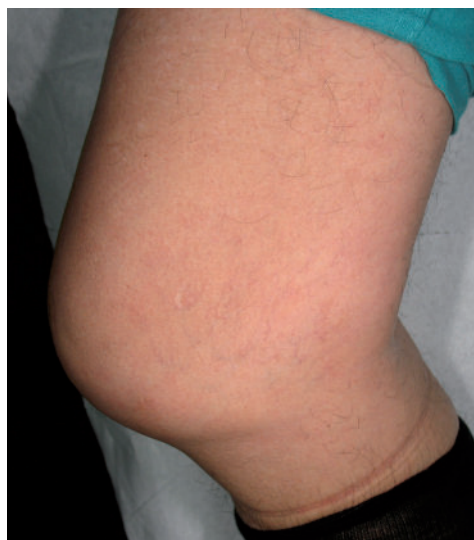

*Three months after a single treatment
(picture taken using polarized light)*

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD

Department of Dermatology

University of Florence, Italy

Lower Limbs Teleangectasias

46-years old

Caucasian female patient

Phototype III

Settings

Fluence 110 J/cm²

Spot Size 5 mm

Nr. of Pulses 2

Pulse Duration 5 ms + 18 ms

Delay 20 ms



Before treatment



Two months after the 3rd treatment

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD


Department of Dermatology

University of Florence, Italy

Lower Limbs Teleangectasias

Settings

Fluence	80 J/cm ²
Spot Size	5 mm
Nr. of Pulses	2
Pulse Duration	5 ms + 12 ms
Delay	20 ms

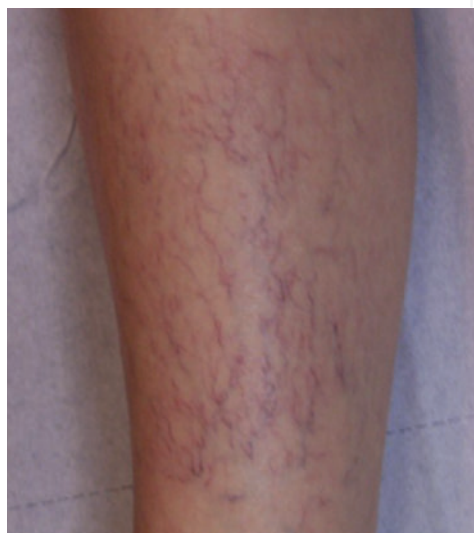

Before treatment



Two months after the 2nd treatment

Courtesy of
L. Mazzi, MD
Specialist in Dermatology
and Venereology Verona, Italy

Lower Limbs Teleangectasias

Settings

Fluence	80 J/cm ²
Spot Size	5 mm
Nr. of Pulses	2
Pulse Duration	4 ms + 10 ms
Delay	20 ms



Before treatment



Six months after the 3rd treatments

Courtesy of
L. Mazzi, MD
Specialist in Dermatology
and Venereology Verona, Italy

Lower Limbs Teleangiectasias

Settings 1st Treatment

Fluence	85 J/cm ²
Spot Size	5 mm
Nr. of Pulses	2
Pulse Duration	7 ms + 12 ms
Delay	20 ms

Settings 2nd Treatment

Fluence	225 J/cm ²
Spot Size	2.5 mm
Nr. of Pulses	2
Pulse Duration	4 ms + 11 ms
Delay	20 ms



Before treatment



Two months after the 2nd treatment

Courtesy of
L. Mazzi, MD
Specialist in Dermatology
and Venereology Verona, Italy

Lower Limbs Teleangectasias

46-years old

Caucasian female patient

Phototype II

Settings


Fluence 110 J/cm²

Spot Size 5 mm

Nr. of Pulses 2

Pulse Duration 5 ms + 18 ms

Delay 20 ms

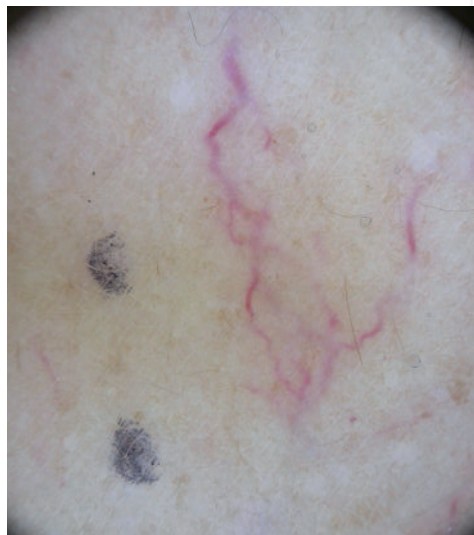

Before treatment (macro)



Immediately after the 1st treatment (macro)

Courtesy of:

P. Campolmi, MD

G. Cannarozzo, MD

P. Bonan, MD

Department of Dermatology

University of Florence, Italy

Hirsutism with Hormonal Anomalies

27-years old
Caucasian female
Phototype V

No possibility to use contraceptive therapy for reducing hairs number.
High risk of paradoxical stimulatory effect due to non-treated hyperandrogenia



Before treatment (macro)



Two months after the 5th treatment

Courtesy of
A. Le Pillouer-Prost, MD
Specialist in Dermatology
and Venereology Marseille, France

Hirsutism with Hormonal Anomalies

42-years old
African female
Phototype VI



Before treatment (macro)



Two months after the 6th treatment

*Courtesy of
A. Le Pillouer-Prost, MD
Specialist in Dermatology
and Venereology Marseille, France*



DEKA M.E.L.A. s.r.l.

© DEKA Rev. 1.0 - All rights reserved